



Exporting from Planmeca

STEP 1. Right click on your desktop, select **NEW** and click on **FOLDER**. Rename the folder to match the patient's name.

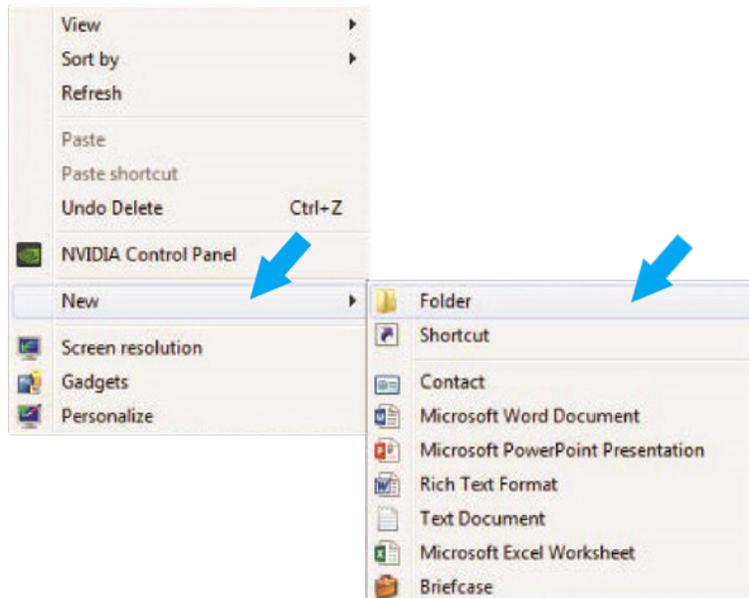


STEP 2. Next, open the patient's scan. At the top of the screen, hover over the drop down menu and in the Output section, select **EXPORT VOLUME**.

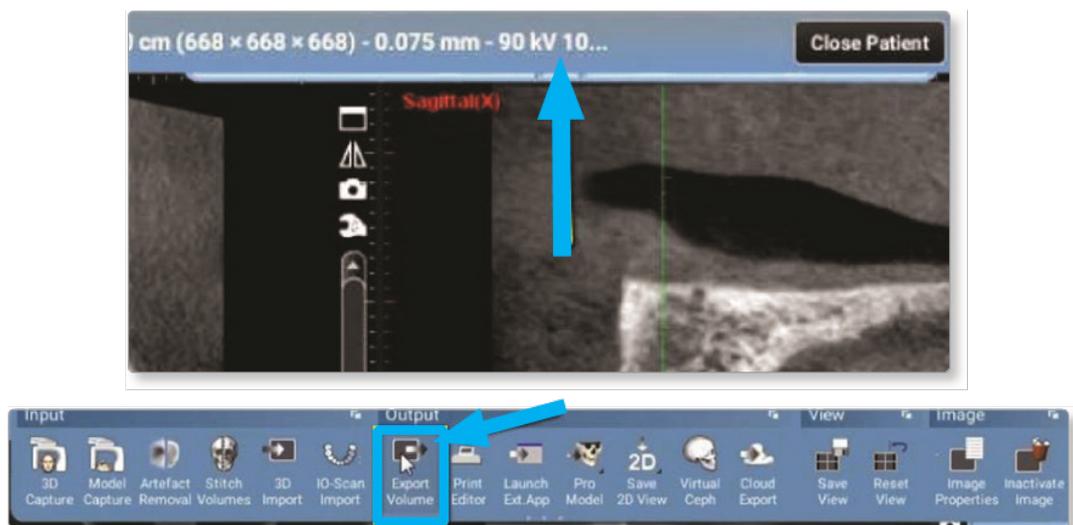


Exporting from Planmeca

STEP 3. Select **SET OF SINGLE FRAME DICOM FILES** (no other option should have to be changed).

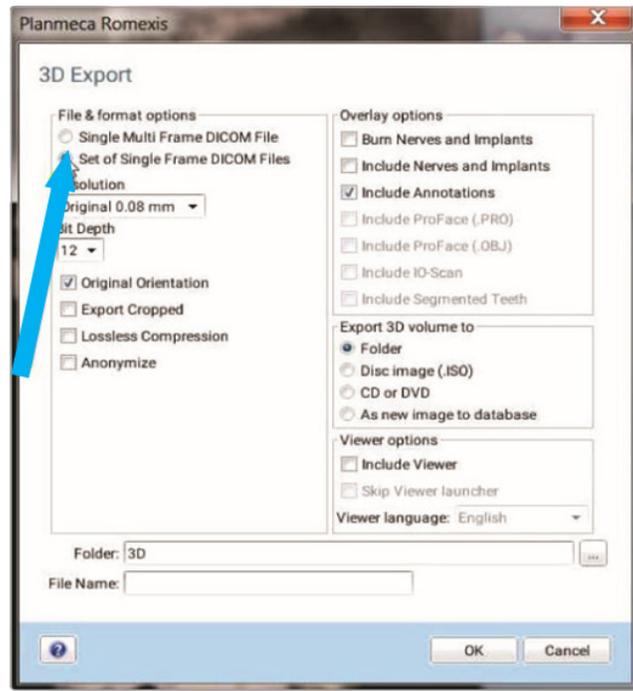


STEP 4. Click on the three dots on the lower right to choose where to export the files.

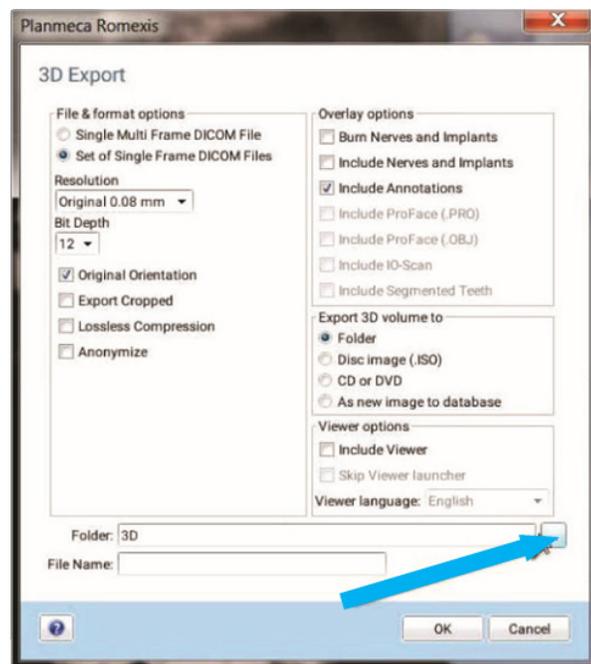



Exporting from Planmeca

STEP 5. Select the patient folder on the desktop and click **OK**.

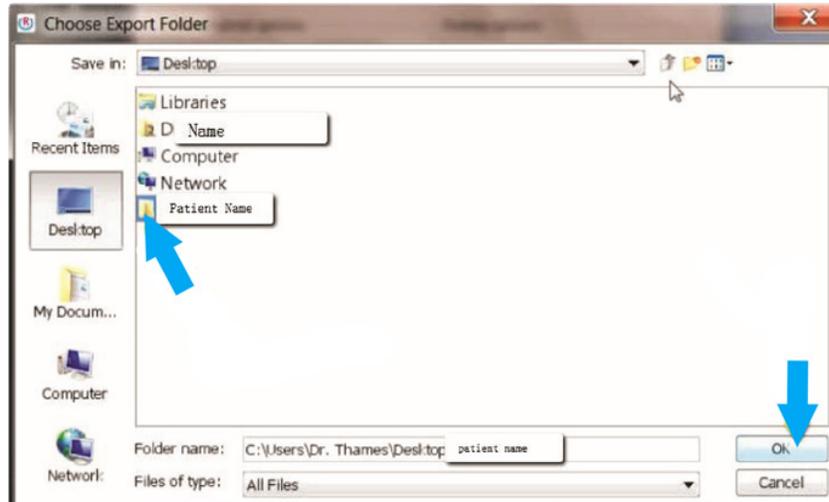


STEP 6. After the pop-up box disappears, under the file name, type **PATIENT NAME**. Click **OK** and the files will begin exporting. Further information for exporting files can be found on this video:

<https://vimeo.com/253317826>

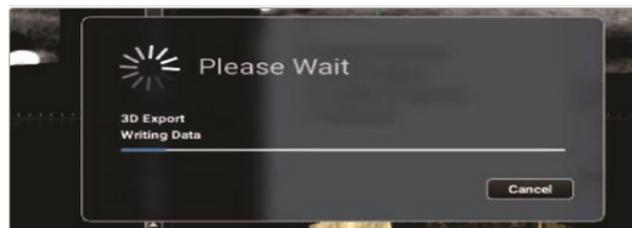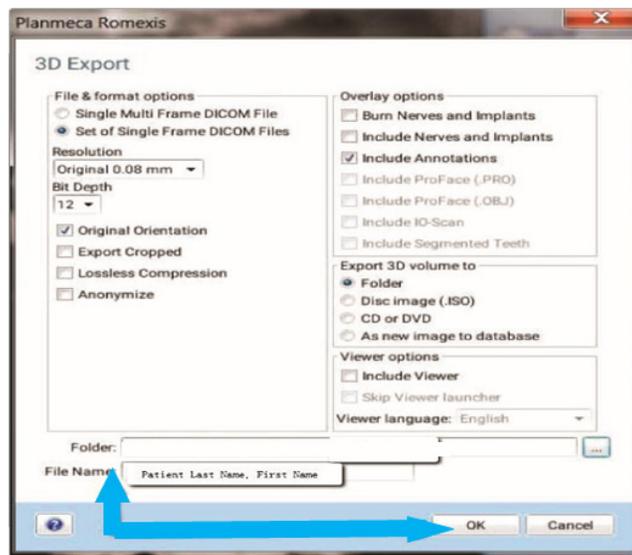


Exporting from Planmeca

STEP 7. After the DICOM exports and saves, go to the patient's folder and verify that the .dcm files were exported correctly (you should see 150 to 450 slices).

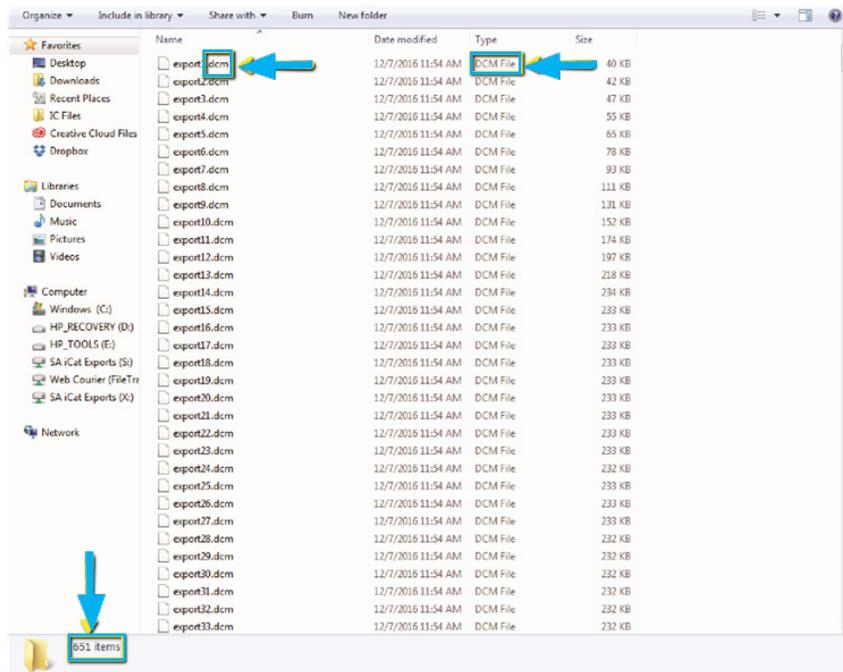


STEP 8. Right click on the patient folder you created on the desktop, select **SEND TO** and **COMPRESSED (ZIPPED) FOLDER**. A duplicate folder will be made with either a zipper or a blue "z."

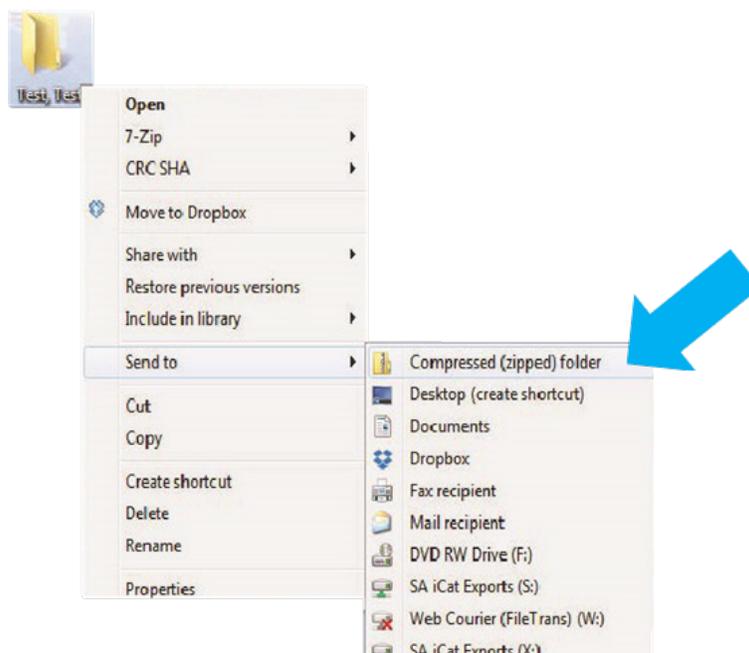

Exporting from Planmeca

STEP 9. You are now ready to upload your file to <https://nsequence.com>. Hover over **SEND A CASE** in the menu. Select **UPLOAD FILES**.

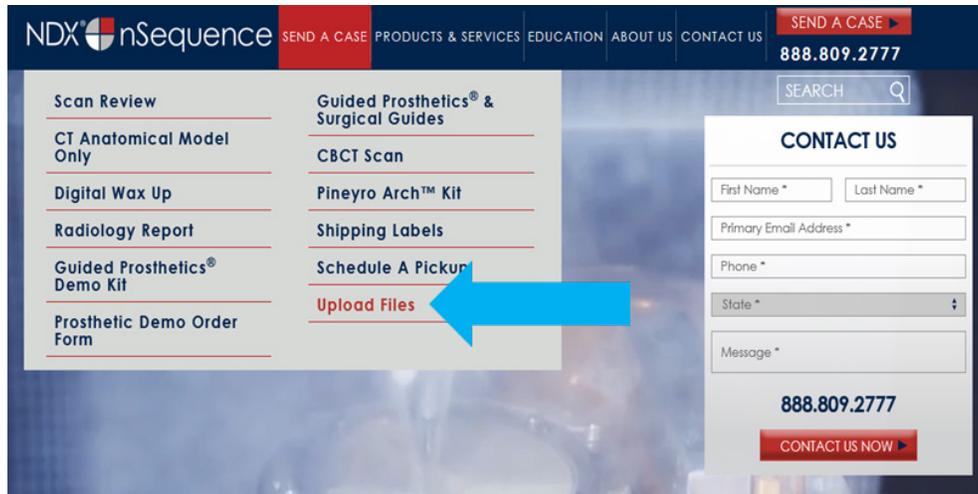

STEP 10. Select **CLICK TO ADD FILES** or Drag and Drop the file in the boxed area.


Prepare Your Data For Upload

Prior to sending us your data, please prepare your data for upload. We can accept anything you would like to send us including photos, DICOM data and other file types related to the cases you're submitting.

1. Maximum total file size = 2GB.
2. If sending large volumes of data, please zip all files prior to sending.
3. Maximum 10 files can be uploaded at a time and upload time is dependent on your Internet upload speed.
4. Make sure you keep the page open until all files have been uploaded, closing the page will result in failed upload attempt.
5. Upload progress will not be shown during upload if you are using Microsoft Edge browser.

We do everything we can to make sure you can upload your files as quickly as possible. If you're experiencing a slow upload time, it may be due to exceptionally large files as well as your Internet Service Provider (ISP) or network connection.

Select Files To Upload

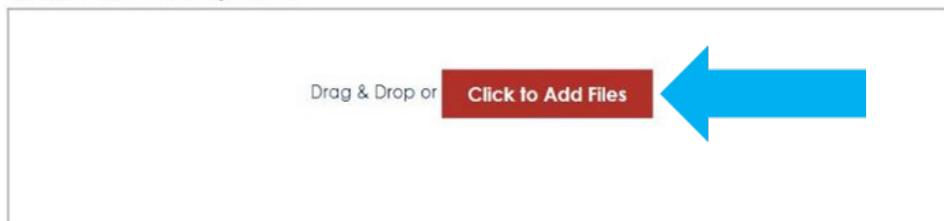

Exporting from Planmeca

STEP 11. Complete the form and click **UPLOAD FILES**. The upload process may take a few minutes.

Email address*
name@example.com

Doctor's First Name*
First Name

Doctor's Last Name*
Last Name


Patient's First Name*
First Name

Patient's Last Name*
Last Name

Notes
Additional case notes, patient info, references, special instructions, etc.

Upload Files

STEP 12. Go back to the menu, hover over **SEND A CASE** and select **Guided Prosthetics[®] & Surgical Guides**.

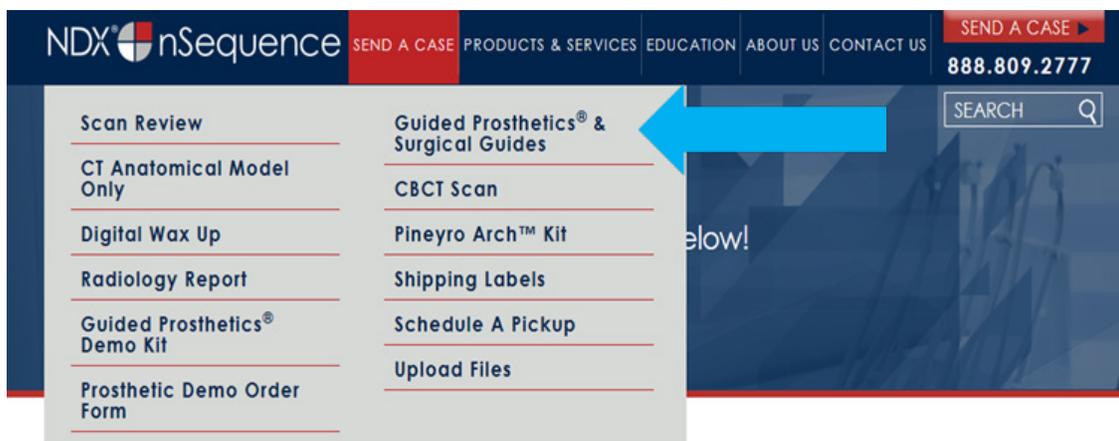


Exporting from Planmeca

STEP 13. Choose the applicable RX order form.

Send a Case > Guided Prosthetics & Surgical Guides

Guided Prosthetics & Surgical Guides

Please select the product you wish to order below.

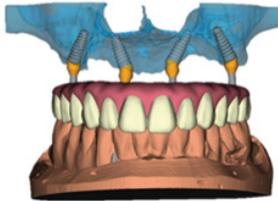

nSequence[®] CT Guided Prosthetics[®] Kit & Workflow

The nSequence Guided Prosthetics Kit includes the following:

- Collaborative Surgical Planning
- Anatomical Model
- Bone Foundation Guide (Reduction Guide)
- Surgical Guide
- Start Bite, Angle Rotation Guide and Prosthetic Bite Guide
- Tissue Gasket
- 2 Long-Term Provisionals
- Surgical Report

[Guided Prosthetics RX Order Form](#)



nSequence CT Surgical Guides

The nSequence CT Surgical Guides include the following:

- Collaborative Surgical Planning
- Surgical Guide
- Optional Temporary Restoration (additional costs will apply)
- Surgical Report

[Surgical Guide RX Order Form](#)



STEP 14. Finally, complete the RX order form and click **SUBMIT**.