

NDX[®] nSequence

nSequence[®] Guided Prosthetics[®]

REQUIRED RECORDS FOR A **FULLY DENTATE ARCH**



Submit an Online RX Form to:
nsequence.com/ct-guided-prosthetics-order

Make sure you provide the following records to nSequence:

1

CBCT Scan WITH
Bite Registration



2

Impressions
(Digital or Analog)



3

Bite Registration
(Digital or Analog)



4

Photos



For detailed information, please visit nsequence.com/virtual-learning-academy »

Topic #2: Best Practices for Record Taking for Full Arch Rehabilitation or contact us at 888.809.2777.

Submit an Online Rx Form

1. Go to nsequence.com
2. Click on Send a Case
3. Click on Guided Prosthetics & Surgical Guides
4. Select Guided Prosthetics RX Order Form

IMPORTANT TO NOTE

We cannot begin planning your case until all records (digital and analog) have been received and validated for quality. Once validated, an online consultation will be scheduled.

Inaccurate/incomplete records will lead to delays.

1

CBCT Scan WITH Bite Registration



1. Patient at desired VDO and MIP with bite in mouth
2. .3 Voxels at 20 seconds
3. 13x13 cm field of view (10x10 cm minimum)
4. Verify scan quality before dismissing patient

2

Impressions (Digital or Analog)



1. Digital full arch impression scan of both arches
2. Scan working arch for a denture
3. Analog full arch (PVS) impression
4. Impression of a working arch for a denture



3

Bite Registration (Digital or Analog)



1. Full arch bite registration



2. Digital image of full arch in occlusion



3. If opening VDO, capture bite at desired VDO/MIP

4

Photos (5 Images)



1. High lip/Smile line
2. Lips slightly parted in a rest position
3. **Retracted** frontal in occlusion
4. **Retracted** right side in occlusion
5. **Retracted** left side in occlusion

For detailed information, please visit nsequence.com/virtual-learning-academy »

Topic #2: Best Practices for Record Taking for Full Arch Rehabilitation or contact us at **888.809.2777**.